Chrisofix[®]
Technology

CHRISOFIX[®] FRACTURE COSTALE
CHRISOFIX[®] RIB TRAUMA

Chrisofix[®]
Technology

Manufacturer :

GIBAUD S.A.S.
73 rue de la Tour - B.P. 78
42002 SAINT-ÉTIENNE CEDEX 1 - FRANCE
www.gibaud.com

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Producer :

Orkrisz Kft on behalf of Chrisofix AG
H-1239 Budapest
Grassalkovich út 272-274
Tel.: +36 1 287 1056
Fax: +36 1 289 0232
www.chrisofix.ch

Chrisofix[®]
Technology

Chrisofix[®]
Technology

[FR]

[EN]

CHRISOFIX[®] FRACTURE COSTALE

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INDICATIONS:

Rib trauma, rib fracture.

CONTRAINDICATIONS AND LIMITATIONS OF USE:

Injury requiring surgery and/or another immobilisation method. Do not use in the presence of open wounds. Protect skin in the event of dermatological lesions.

This medical device should only be used in the recommended indications.

PRECAUTIONS FOR USE:

The splint will be applied the first time by a qualified fitter, according to medical recommendations.

Medical recommendations related to duration of wear and product readjustments must be followed closely.



Never cut the splint. The exposed aluminium in the fabric wrap could cause an injury.

FEATURES:

- Can be adjusted in 1 minute, without water or heat.
- Light, comfortable, non-allergenic and latex-free.
- Perforated (breathable).
- Radiotransparent.
- Simple use.

FITTING INSTRUCTIONS:

- 1 - The skin must be shaved and degreased before the brace can be applied.
- 2 - Fit the plate to the shape of the thorax so the arrows are parallel to the rib direction and the fractured rib is well surrounded.
- 3 - Remove the protective film around the plate.
- 4 - Remove the adhesive film from the plate and attach the plate in the configuration determined above.
- 5 - Remove outer transparent sheet protection.

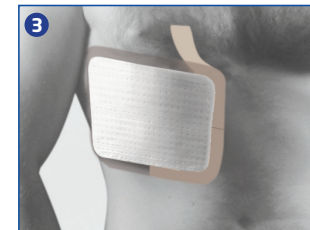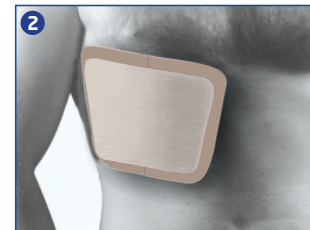RECOMMENDATION:

Even if your pain is gone, do not stop wearing the splint unless instructed to do so by your doctor.

CARE AND MAINTENANCE:

Store in a clean environment, protected from excessively high temperatures, moisture and light.

SIZES AND MEASUREMENTS:

One size fits all.

Splint size	1
-------------	---